



Date : 01/26/2015

Dear Health Care Professional:

This document provides information on the material and functional compatibility of Olympus devices with the selected reprocessing method and/or chemistries listed at the end of this letter. Please refer to the product-specific instruction manual for detailed information on device reprocessing.

DEFINITIONS

Compatible: Fully compatible. Device material compatibility and functional performance was validated.

See Remarks: Device material compatibility and functional performance validated with limitations. See remarks for details.

Not Compatible: Device was not materially compatible and/or functional performance of the device was not maintained.

N/A: Testing was not performed or the results of testing were not conclusive.

NON-OLYMPUS REPAIRS

Instructions provided in this document regarding material compatibility are not valid for Olympus devices repaired by a non-Olympus facility. Olympus repairs devices to manufacturer's specifications using original equipment manufacturer's (OEM) materials. The use of non-OEM materials to repair an Olympus device may affect the material compatibility of the device with certain reprocessing chemicals or methods. In the event that your device has been repaired by a non-Olympus facility, please contact that repair facility for instructions regarding material compatibility.

WARRANTY

Nothing contained in this letter alters, extends, or modifies in any way the authorized Olympus warranty applicable to each device or instrument.

If you have any questions regarding the compatibility of Olympus products, contact your local Olympus sales representative or the Olympus Technical Assistance Center at 1-800-848-9024. Thank you.

Sincerely,

Mary Ann Drosnock, MS, CIC, CFER
Manager, Infection Control
maryann.drosnock@olympus.com

OLYMPUS AMERICA INC.
3500 Corporate Parkway
P.O. Box 610
Center Valley, PA 18034-0610
Telephone: (484) 896-5000
www.olympusamerica.com
Page 1 of 4

Product Family : Duodenoscope (TJF & JF)

Product Name : TJF-Q180V

**Reprocessing Method /
Chemistry**

Remarks

Detergent	Disinfection of endoscopes should be performed according to the labeling of ALDAHOL solution.
FlexClean 895	Compatible
Aldahol	Compatible
Glutaraldehyde (2 - 3.5%)	Compatible
Cidex OPA Room Temp (20°C for 12 minutes)	Slight cosmetic changes (grayish) may be observed with reprocessing tubes (e.g., injection tube) and adhesive. However, these products are functionally compatible with Cidex OPA when used according to its label condition.
Metricide OPA Plus	Olympus has not performed testing of Metricide® OPA Plus Solution (Metrex Research Corporation) with Olympus endoscopy equipment. Although Olympus recommends use of only those products that have passed our internal testing, the lack of a compatibility statement from Olympus pertaining to Metricide OPA Plus Solution in no way prejudices its compatibility with Olympus endoscopy equipment, nor does the use of an untested product automatically void the Olympus standard warranty. However, Olympus cannot accept any responsibility for damage attributable to the use of untested products, just as we cannot be responsible for damage that results from the improper use of an approved product. Olympus has tested the compatibility of Cidex® OPA Solution (Advanced Sterilization Products). Metricide OPA Plus Solution is labeled and marketed as a high-level disinfectant/sterilant for medical devices, similar to Cidex OPA Solution. The instructions for use indicate that the contact time at room temperature is equivalent to Cidex OPA Solution, but the product has a slightly higher concentration of ortho-phthalaldehyde. The rinse requirements for Metricide OPA Plus Solution are also slightly different than Cidex OPA Solution, with Metricide OPA Plus Solution requiring a greater minimum volume of water for each rinse. In the absence of a formal device compatibility study, Olympus is unable to make a statement regarding the compatibility of Metricide OPA Plus Solution with Olympus endoscopy equipment. However, Olympus is unaware of any reports or instances of material compatibility-related complaints from Olympus users regarding this product. Please follow the instructions for use for any liquid chemical germicide, and in the case of OPA solutions, please pay particular attention to the rinsing instructions and any contraindications for reprocessing specific types of flexible endoscopes. For information on the compatibility of Cidex OPA solution with Olympus endoscopy products, please select Cidex OPA in the interactive reprocessing guide.
Steam (132°C at 5 min.)	Not Compatible
Steam (132°C at 4 min.)	Not Compatible
Steam (134°C at 3 min.)	Not Compatible
Steam (135°C at 20 min.)	Not Compatible
ETO 100%	Compatible
Sterrad 50	Not Compatible
Sterrad 100S	Not Compatible
Sterrad 200	Not Compatible

OLYMPUS AMERICA INC.
3500 Corporate Parkway
P.O. Box 610
Center Valley, PA 18034-0610
Telephone: (484) 896-5000
www.olympusamerica.com

Sterrad NX (Use Advanced Cycle for flexible endoscopes and accessories: Use Standard Cycle for cameras and rigid products: Use Standard or Advanced C	Not Compatible
Sterrad 100NX (Use Flexible Cycle for flexible endoscopes and accessories: Use Standard Cycle for cameras and rigid products)	Not Compatible
Resert XL	N/A
Rapicide PA (DSD Edge)	N/A
S40 Sterilant Concentrate (System 1E)	N/A
CIDEX OPA-C Concentrate (Evotech)	N/A
TSO3 Ozone Sterilizer (Sterizone) 125L cycle	N/A
Vaporized Hydrogen Peroxide (V-Pro 1)	N/A
TSO3 Ozone Sterilizer (Sterizone) 125L+ Cycle #1	N/A
TSO3 Ozone Sterilizer (Sterizone) 125L+ Cycle #2	N/A
TSO3 Ozone Sterilizer (Sterizone) 125L+ Cycle #3	N/A
Reliance DG Dry Germicide	N/A
Aldahol (in the OER-Pro)	Compatible
Acecide-C (in the OER-Pro)	Compatible
Ethyl Alcohol (Ethanol)	Compatible
Isopropyl alcohol (Isopropanol)	Compatible

Sodium Hypochlorite (Bleach Solution)	N/A
V-Pro 1 Plus (Lumen Cycle)	N/A
V-Pro 1 Plus (Non-Lumen Cycle)	N/A
V-Pro Max (Flex Cycle)	N/A
V-Pro Max (Lumen Cycle)	N/A
V-Pro Max (Non-lumen Cycle)	N/A
STERRAD 100NX (Duo Cycle)	N/A
Rapicide OPA-28 High-Level Disinfectant	N/A
Sani-Cloth Plus Disinfectant Wipe	N/A
